
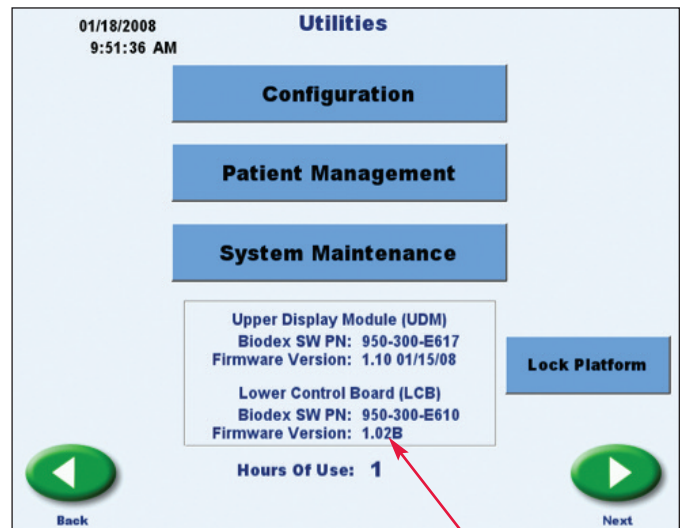


# Instructions for Software

# VERIFICATION

FOR BALANCE SYSTEM™ SD AND BIOSWAY™ WITH 8" DISPLAY FOR UPGRADE TO 12" DISPLAY

## Verify Lower Board Version

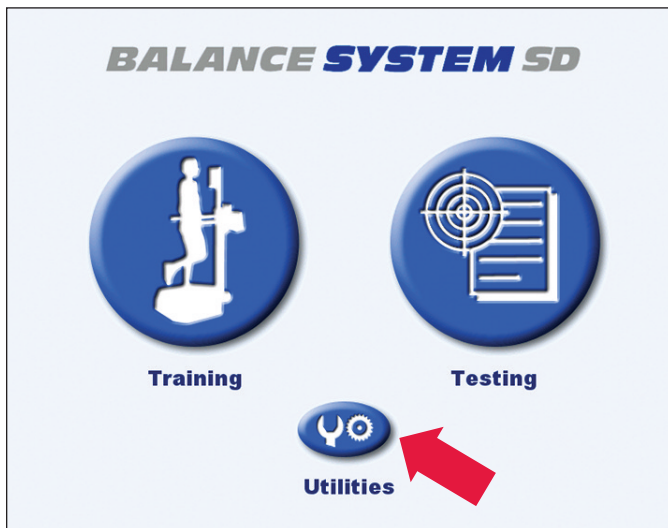

Sample shows Lower Board Version 1.02B

To verify the current Lower Display Board version installed on your present system:

1. Select <Utilities> button on Home screen.
2. Verify Lower Control Board (LCB) version, last line.

**Applies to Models:**  
**BALANCE SYSTEM SD & BIOSWAY**

950-300 Balance System SD  
950-330 BioSway

**NOTE:** Versions 1.02B and below require a service visit and the installation of a new strain gauge board 950-300-E810.

**NOTE:** Versions 1.03 and above can be upgraded successfully by simply installing display kit 950-440-K900.



### Need Help?

Contact Biodex Service Support:

Phone: 1-631-924-9000, ext. 2120

Fax: 1-631-924-8355

Email: [servicesupport@biodex.com](mailto:servicesupport@biodex.com)

# BIODEX

Biodex Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704, Tel: 800-224-6339 (Int'l 631-924-9000), Fax: 631-924-9338, Email: [info@biodex.com](mailto:info@biodex.com), [www.biodex.com](http://www.biodex.com)